


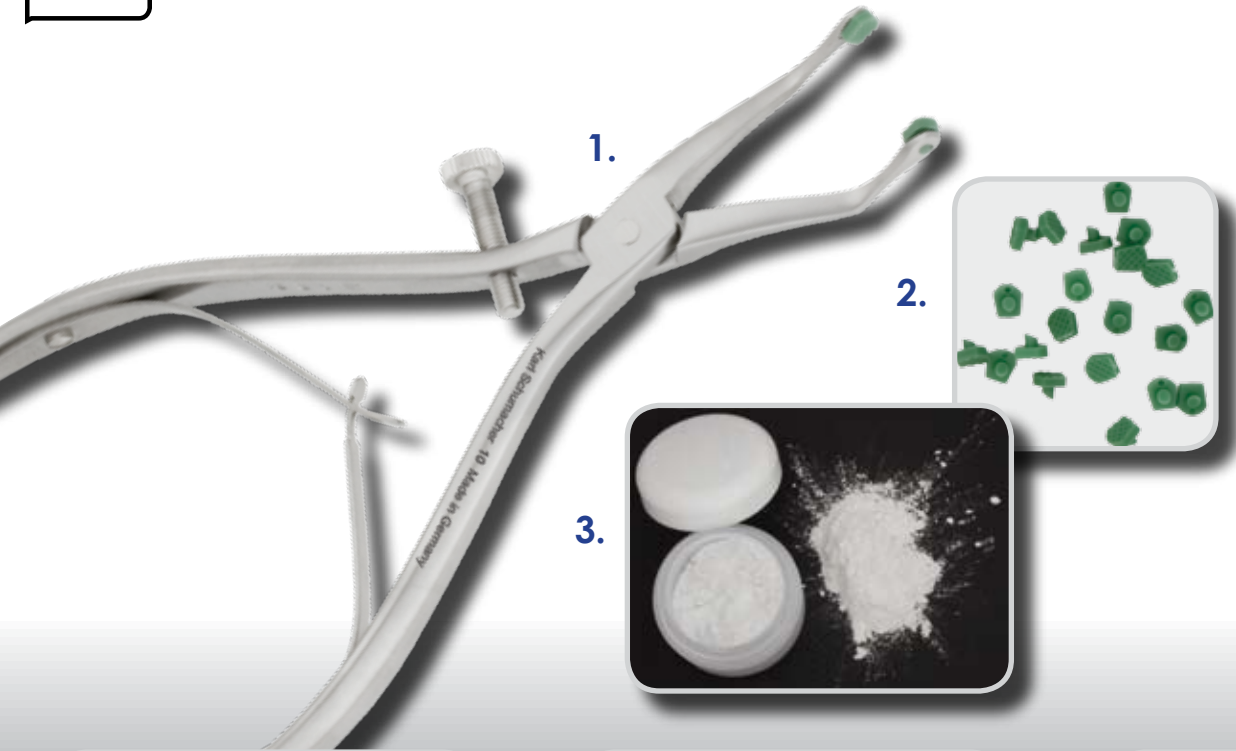

# Crown Tractor #10, with Set Screw, 10 Pair of Green Inserts and 1 oz. Diatrac Powder

- 1. **CRE0010** crown tractor
- 2. **CRE0010I** replacement inserts
- 3. **CRE0010D** replacement diatrac



The Crown Tractor #10 is the most versatile crown remover we carry. With advanced features like a spring handle, an adjustable set screw and autoclavable silicon pads, the Crown Tractor is able to safely remove all of your crowns. The included Diatrac powder is powerful enough to easily remove permanently cemented crowns, yet will not scratch the porcelain. Also, by removing the silicone pads, the Crown Tractor can be used to remove temporary crowns with the heavily serrated metal tips.



Position and grasp the crown lightly just below the height of contour. Tighten the set screw so it just touches the opposing handle.



Remove the instrument from the crown and back the set screw out 1/2 turn. This will allow the instrument to grasp the crown firmly without crushing or chipping it.



A small amount of Diatrac powder should be placed in a dappen dish. Moisten the tips of the Crown Tractor and lightly coat them. This is **IMPORTANT**, as the Diatrac powder provides optimal traction.



Reposition the Crown Tractor onto the crown. A very firm grip can be applied as excessive force is absorbed and distributed by the pads. On release, the crown will be protected from crushing damage by the set screw.